

LIBERTY 300 3G MEDICAL ALARM INFORMATION SHEET

HOW DOES IT WORK?

The Liberty 300 Alarm is designed to assist people to maintain their independence in the home. When the Liberty 300 is activated in an emergency, it dials the 24-hour response centre putting the user in contact with a response operator, who will aid the client by assessing the situation and contacting the necessary responders to assist.

Each Liberty 300 is supplied with a personal radio trigger that can be worn on the wrist or around the neck as a pendant, allowing the user to activate the alarm from anywhere in the house, without having to struggle to a phone.

The pendant can be fitted with our easy-press adapter, a silicon case that improves the sensitivity of the alarm button for those with limited hand or finger movement. Our Liberty 300 fall pendant can be paired with the alarm, automatically generating a call for assistance if a fall is detected.

Alternatively, the alarm can be activated by the red help button on the base of the alarm unit. Once an alarm call is raised, the user can then speak hands-free with the response operator who will summon the appropriate help.



WHO IS IT FOR?

The high degree of flexibility offered by the Liberty 300 alarm means a wide variety of users can benefit from the state of the art technology, including:

Older people who are living with a disability

Individuals who have been discharged from hospital and require additional support

People working alone

Those with long-term conditions

Available sensors and accessories:

Pendant, neck chain and wrist strap (included)

Fall detector pendant

Easy-press adaptor

SERVICE SUPPORT FEATURES:

Radio reliability – the AS4268.2012 radio receiver ensures that signals from sensors are reliably received

Long range pendant – the pendant has a range of 50 – 100 metres from the alarm unit in open air

Water-resistant pendant – the pendant is also water resistant up to one metre, enabling it to be worn in the bathroom where most falls occur

Programming and installation features:

Plug and play registration – telecare sensors can be assigned quickly and easily, reducing installation times

Local and remote programming – via Liberty 300 programming software